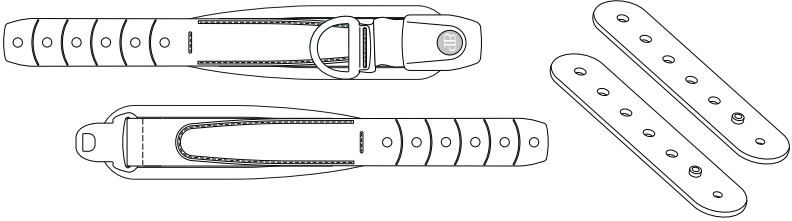
Pelvic Positioning Belt & Belt Extender

Art Nos. EB205, EB235, EB275, EB285 & EBE100

INSTALLATION AND USER'S INSTRUCTIONS

These instructions provide important information for the safe use and maintenance of the Bodypoint Evoflex Belt and optional Evoflex Belt Extender. Give these instructions to the user or their caregiver and review them to ensure that they are understood.

INTENDED USE:

A positioning belt to increase sitting stability, maintain or correct posture, and maintain a safe seated position in a wheelchair, stationary positioning chair, or commode chair.

DEVICE DESCRIPTION:

A pelvic positioning belt. When applied by a trained healthcare professional, this device can be used independently for support and positioning.

CONTRAINDICATIONS:

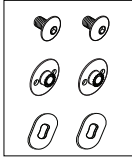
This product is NOT intended for use as a transportation safety device, as a personal restraint device, or in any other application where its failure could result in injury. Misuse of this product is unauthorized and unsafe.

WARNINGS:

1. This belt must be worn snugly fitted across the pelvis or thighs at all times.
 - If it is too tight, it can increase pressure across the body.
 - If it is too loose, it can allow the user to slip down and may create a risk of strangulation.
 - Accidental release of this product can allow the user to fall forward, slip down or fall from the wheelchair.
 - A user's inability to self-release can be hazardous if the user slips down or is trapped in the chair in an emergency.
2. If the user's physical or cognitive abilities prevent them from safely operating this belt, a caregiver must be present at all times during its use. Ensure that all caregivers know how to correctly tighten and unfasten the product.
3. This product may change the way a person sits. Users must continue to practice regular pressure relief activities and skin integrity checks, not only where this product contact. If increased skin redness or irritation occurs, discontinue use and consult your physician or seating specialist. Failure to do so may result in serious injury, such as pressure ulcers.



PACKAGE CONTENTS

Evoflex pelvic positioning belt with HKA11 Hardware Kit

HKA11 Hardware Kit

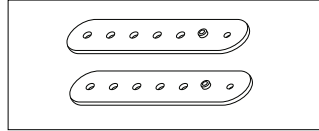
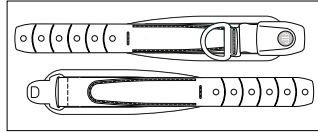
- (2) M6 X 8 Screws
- (2) M6 Low-profile Flange Nuts
- (2) Wave Spring Washers

Art. Nos. ending in -EBE also include:

- (2) Evoflex Belt Extenders
- HKA11 Hardware Kit

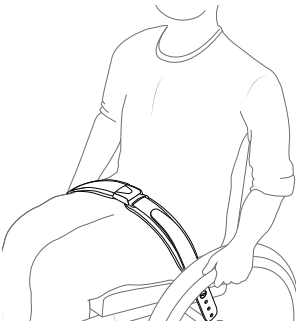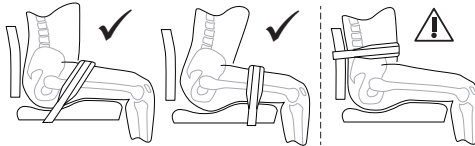
INSTALLATION

1. Based on clinical goals, determine the most effective position and angle for the belt. Determine whether putting pull strap on left or right side is best for user.
2. Locate desired mounting points on the seating system or wheelchair frame.
3. Test the belt position and length by placing it over the user, leaving 5-10 cm [2-4"] slack at the buckle. Note the numbered hole in the plastic strap which most closely matches the desired mounting point on the wheelchair.
4. Using the included hardware kit, attach the belt to rigid seat base or wheelchair frame using optional Bodypoint belt mounting hardware, if needed (see Mounting Options).
5. After verifying the belt fits correctly, the ends of the plastic straps can be shortened by cutting along the curved lines with a saw or heavy-duty scissors. It is recommended to leave at least one extra hole to allow for minor adjustment.

ADJUSTMENT

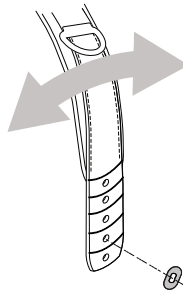
When the belt is properly adjusted and tightened, it should fit snugly, with all slack removed, so that the user is secure. Use the pull strap for daily adjustments, such as to accommodate changes in clothing. If additional length adjustment is necessary to achieve a good fit or the belt needs to be removed for cleaning, this can be done at the end strap.

CONTROLLING ROTATION


Where needed to increase tension and control rotation, use included wave spring washer. Omit wave spring washer where tension is sufficient without it.

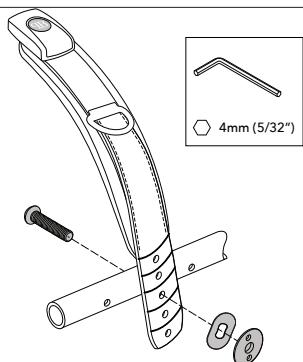MOUNTING OPTIONS


DIRECT MOUNTING TO FRAME
(Requires an M6 screw (not included)
medium-strength class 8.8)

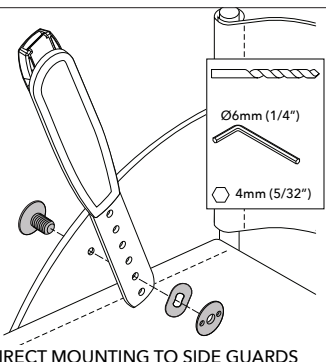
Choose the screw length according to the section of the wheelchair frame it must pass through. Ensure screw does not protrude beyond the low-profile flange nut, which can create a risk of injury to the user.

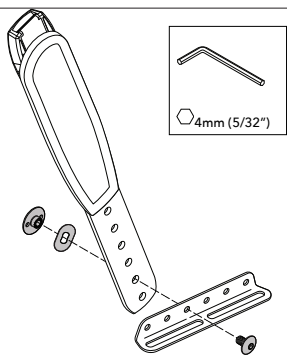
DIRECT MOUNTING TO SIDE GUARDS
(Use fasteners from HKA11 Hardware Kit)
Drill 6mm (1/4") hole or use existing hole/
slot to attach belt using included fasteners.

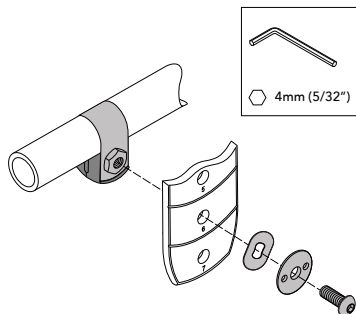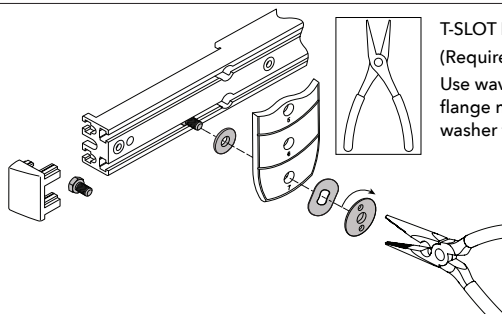SEAT MOUNTING BRACKET
(Requires Art. No. HW200-20 or -90)

BAND CLAMP OR FRAME CLAMP
(Requires Art. No. HW320 or HW310)
Attach clamp to seat tube. Attach belt
using the M6 x 12 or M6 x 16mm button
head screw included with the Band Clamp
or Frame Clamp. **DO NOT USE** the M6 x 8
flange head screw included in the HKA11
Hardware Kit—it is too short and can result
in loosening.

T-SLOT FASTENER KIT
(Requires Art. No. HKA6-12 or HKA10-14)
Use wave spring washer and low-profile
flange nut from HKA11 with bolt and
washer from T-Slot Fastener Kit.

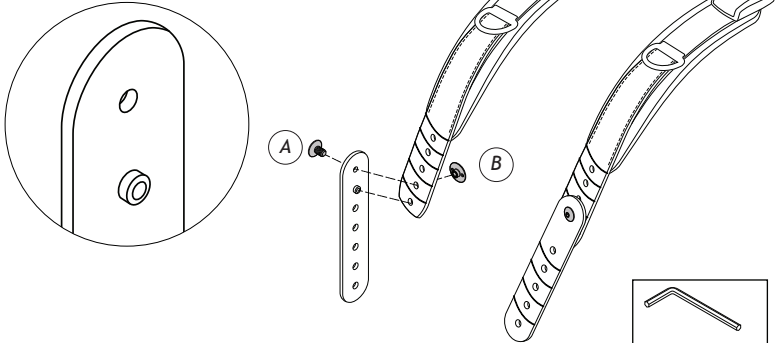
EVOFLEX BELT EXTENDER (OPTIONAL)

Attach Belt Extender to Evoflex Belt using included HKA11 Hardware Kit. Install M6 x 8 screw through top hole of Belt Extender as shown in one of the following two methods.

-   • (2) M6 X 8 Screws (A)
-   • (2) M6 Low-profile Flange Nuts (B)
-   • (2) Wave Spring Washers (not required)


TO PROHIBIT BELT ROTATION:

Pin facing in

TO ALLOW BELT ROTATION:

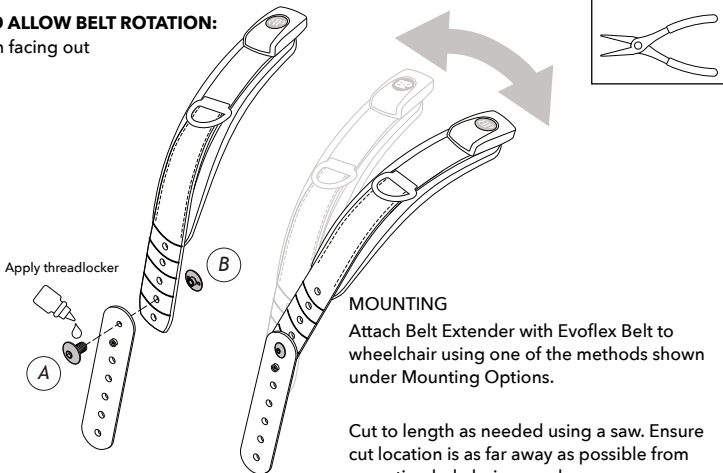Pin facing out


MOUNTING

Attach Belt Extender with Evoflex Belt to wheelchair using one of the methods shown under Mounting Options.

Cut to length as needed using a saw. Ensure cut location is as far away as possible from mounting hole being used.

For more information on Bodypoint mounting hardware visit:
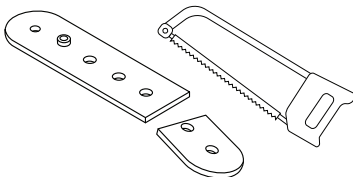
<https://app.bodypoint.com/category/hardware>


OPTIONAL ACCESSORIES


The following optional accessories are available for specific Evoflex Belts.

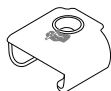- EB205 with Push-Button Buckles is compatible with FS018C / FS019C Security Buckle Covers (Ø6mm button) and FS018D / FS019D Reduced Access Buckle Covers (Ø14mm button).

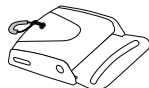- EB235 with Metal Push-Button Buckle is compatible with FS029C Security Buckle Cover.

- EB285 with DuraLatch™ Buckle is compatible with FS200 Pull Loop.

SAFETY CHECKS

When properly positioned in wheelchair, have user lean forward and side-to-side to check fit. Check for:

1. Normal operation of buckles and adjustment straps.
2. Comfort - if any rubbing occurs, relocate straps.
3. Position - if too high or too low, adjust mounting location.
4. Interference with other devices - Relocate straps as necessary to clear armrests, accessory pads, or feeding tubes.

To ensure user safety, this product must be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found in the buckles, mounting points, straps, or stitches, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons – health and safety depend on it!

MAINTENANCE, INCIDENTS, AND DISPOSAL

CLEANING: Machine wash, hot, 60°C (140°F). Do not bleach. Drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

SCRAP/DISPOSAL: Product is made of materials which can be disposed of safely without special precautions at the end of its useful life.

SERIOUS INCIDENT: If a serious incident occurs related to the use of this product, it should be reported to the manufacturer (Bodypoint) and the local Competent Authority.

WARRANTY: This product carries a limited-lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims. For more information on Bodypoint products, and a list of distributors outside the USA, go to www.bodypoint.com

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